FÉLICITATIONS

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CAUTION

Snoring may be a sign of sleep apnea.

4% of men and 2% of women suffer from sleep apnea.¹

If you or your relatives have noticed breathing pauses during your sleep, it is recommended you consult a doctor.

SYMPTOMS OF SLEEP APNEA

DURING THE DAY	AT NIGHT
• Mood alteration • Excessive sleepiness • Lack of energy • Headaches upon awakening • Attention, concentration and memory disorders • Depressed mood • Sexual issues	• Loud snoring • Breathing pauses noticed by relatives • Frequent awakenings, sensation of choking • Restless sleep • Frequently waking up at night to urinate • Night sweats

Ref.: ¹ Young et al. (1993).

RISKS RELATED TO SNORING

Snoring causes light damage to the pharyngeal walls. The resulting scar tissue reduces their suppleness and narrows the airways. Consequently, upper airways age faster than they should which, in itself, might increase snoring.

Age and weight gain also have an effect on snoring.

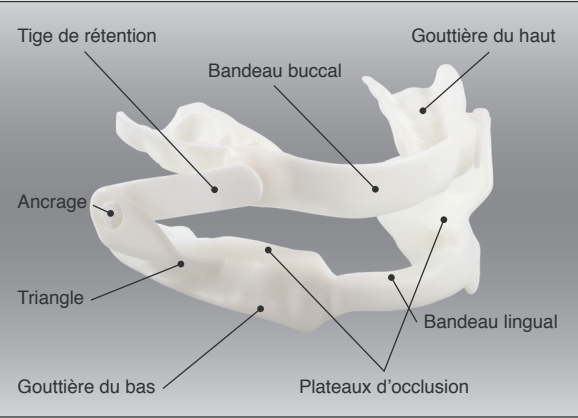
If untreated, snoring may worsen and evolve into more serious problems and diseases such as:

- Excessive daytime sleepiness (the tendency to sleep involved in the risk of car and work accidents)
- High blood pressure
- Type 2 diabetes
- Morning headaches
- Myocardial infarction
- Glaucoma
- Hearing loss
- Decreased libido
- Relationship problems
- Divorce
- Sleep apnea

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WARRANTY

Your Panthera Anti-Snoring Device is guaranteed 3 years against any manufacturing or material defects.

Keep your warranty card. It indicates your file number, the name of your dental practitioner and the date of manufacturing of the device.

In case of failure, it will allow us to quickly replace your device.

For more information, visit www.pantherasleep.com

Since the Panthera device is fixed on the stronger back teeth, tooth movements are rare and minor. If you notice some changes, please consult your dental practitioner.

The Panthera device should not alter occlusion in a permanent manner. If you experience this effect, rapidly consult your dental practitioner.

HELPFUL MORNING EXERCISES

The Panthera Anti-Snoring Device is fixed on the teeth and jaw protrusion stresses the temporomandibular joints. It is thus normal if you have some difficulties to place your jaws in their usual position, at least initially.

It is suggested to perform the following exercises to help reposition the lower jaw:

- Move the chin backward and close molars together while biting. Repeat 5 to 10 times.
- Eat breakfast or chew sugar-free gum; it will help jaws to return to a normal bite position.

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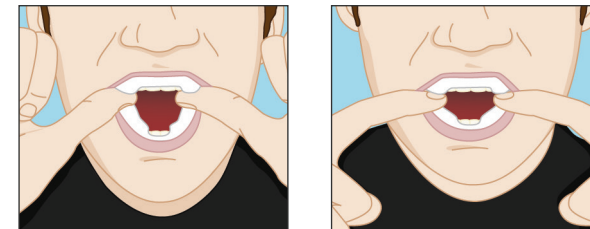
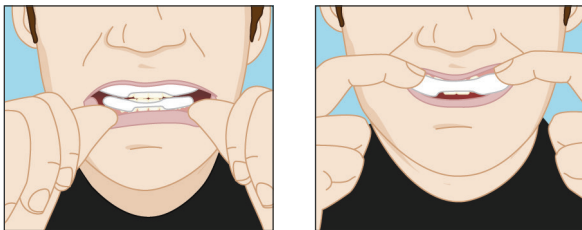
POSSIBLE INCONVENIENCE

You may feel discomfort or stiffness after wearing the device at first; this is normal. However, if the PAIN is INTENSE, it is recommended to immediately stop wearing the device and consult a dental practitioner.

OCCASIONNAL AND TEMPORARY EFFECTS MAY INCLUDE:

- Increased salivation at the beginning, but it should cease over time.
- Tooth sensitivity that should stop after a few minutes and diminish over time.
- Stiffness or pain in the joints of the jaw that should cease after 30 minutes and diminish over time. If the pain continues or increases, consult a dental practitioner.
- Impression of a change in the positioning of the jaw or difficulty to replace lower jaw in its normal position; these effects should vanish after approximately 30 minutes (see "Helpful Morning Exercises").
- Impression of a discrepancy when closing the mouth while wearing the device. You should feel that both sides of the upper and lower splints come together at or about the same time in a balanced manner. Otherwise, the retainers may need a slight adjustment, please consult a dental practitioner.

PRECAUTIONS

The Panthera Anti-Snoring Device is for adults and must be used exclusively during sleep: at night or when taking a nap.

Check the condition of the device on a regular basis. It is recommended to consult a dental practitioner for an annual check-up.

The development of new aching might be caused by an improper adjustment of the device, it is then recommended to stop wearing the device and consult a dental practitioner.

The device is manufactured using high quality bio-compatible material, but rare cases of allergies or inflammation may occur. You must stop wearing the device and consult your dental practitioner.

The device is resistant to temperature changes, but it must be handled at room temperature. So if the device has been left in the car in severe cold, it must return to room temperature before it can be handled.

Note: Wearing a mandibular advancement device is not recommended for people suffering from osteoarthritis of the temporomandibular joints.

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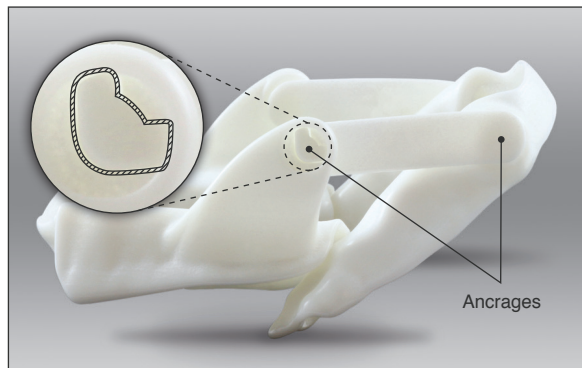
THE NOVADENT CLEANSING SOLUTION REMAINS ACTIVE FOR 7 DAYS

A CURSOR INDICATING THE DAY

REMINDS YOU WHEN THE SOLUTION MUST BE CHANGED

WARM WATER +
NOVADENT ORIGINAL
OR NOVADENT IP

WARNING

Even if the NovaDent Original and NovaDent IP solutions are chlorine-free, **avoid contact with dyed fabrics or materials such as clothing, carpets, rugs, towels, etc.**

Make sure you always keep the bottom of the soaking bath **dry** since the solutions can stain countertops.

Note: Please read the manufacturer's instructions.

USING NOVADENT ON A DAILY BASIS

TO KEEP YOUR DEVICE WHITE

- Pour a 5 gram pouch of NovaDent Original or IP in 3/4 cup (185 ml) of water. You may use:
 - Bottled water (or tap water left to stand for a few hours)
 - Warm water for a quick dissolution (or water at room temperature).
- Keep the solution at room temperature in a **tightly closed container**.
- Leave the device submerged in the solution all day.
- Remove the device from the soaking bath. Rinse thoroughly with warm water. If necessary, clean carefully using a toothbrush.

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MAINTENANCE AND STORAGE

Along with your device, you will receive the following items: a storing case, a carrying pouch, a soaking bath and a pouch of NovaDent cleansing solution.

When traveling, place the device in the storing case.

Keep away from children.

Avoid leaving the device on the night table.

Dogs love it.

Submerge the device in the NovaDent cleansing solution after use (see “Using NovaDent on a Daily Basis”). NovaDent pouches are available in most drugstores. As a complement, you can also clean the device using a toothbrush and toothpaste.

Rinse the device thoroughly with warm water to remove any traces of the NovaDent cleansing solution before insertion.

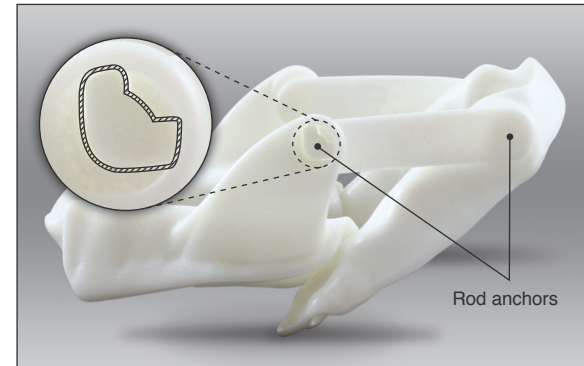
TO CHANGE RODS

To place the rods in their anchors or to remove them, flip over the upper retainer (180 degrees). This positions the retaining rod so it can detach from the rod anchor. Once positioned, push retaining rod through the hole to detach.

To increase the advancement of the lower jaw, insert a shorter rod, i.e. one indicating a smaller number (example, change rods number 30 for rods number 29).

To reduce the advancement of the lower jaw, insert a longer rod, i.e. one indicating a higher number (example, change rods number 29 for rods number 30).

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ADVANCEMENT PROTOCOL

Originally, the advancement of your device will be of approximately 70% of your maximum protrusion. The mandibular advancement must be achieved progressively and accordingly to your tolerance limit so it does not result in joint trauma.

We recommend a progressive mandibular advancement, 1 mm at the time.

It is recommended to use rods of a given length during 2 to 3 weeks before using shorter rods.

According to your comfort and tolerance, the lower jaw will be moved forward until snoring is reduced or suppressed, or until your maximum protrusion capacity has been reached.

Follow the dental practitioner guidelines.

TO REMOVE THE DEVICE

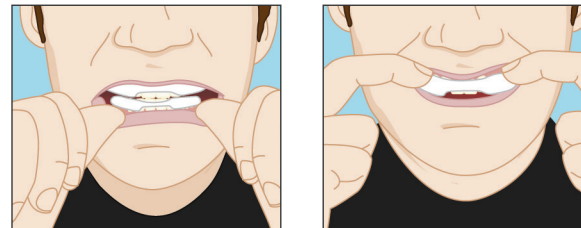
First, unclip the lower retainer.

To do so, place thumbs on the sides of the lower retainer, near the gum, in front of the cuspids, and lift the device.

Afterwards, place fingers on the sides of the upper retainer, near the gum, and pull down the retainer.

Remove the device from mouth, rinse it and submerge in the NovaDent cleansing solution.

NEVER PRESS OR PULL on the retaining rods.

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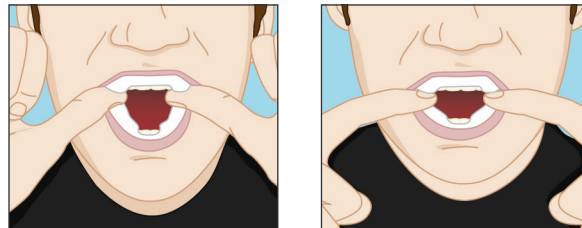
TO INSERT THE DEVICE

Brush teeth and rinse device thoroughly with warm water in order to eliminate any traces of the NovaDent cleansing solution.

Place the Panthera device in the mouth; position the upper retainer (the one with no triangle) on the upper teeth. Press with fingers on the retainer so it fits perfectly on the teeth.

Move the lower jaw forward in order to position the lower retainer on the lower teeth. Press with fingers on the retainer to put it in place.

In some cases, it might be easier to insert the lower retainer first. If so, after inserting the lower retainer, you must move the lower jaw forward in order to insert the upper retainer on the upper teeth.

Note: Initially, it may be easier to insert the device in front of a mirror.

THE ANTI-SNORING DEVICE IS RECOMMENDED FOR:

- Snoring
- Mild or moderate sleep apnea
- Severe sleep apnea when one refuses CPAP
- Alternatively with CPAP (practical when traveling)

Ref.: Canadian Thoracic Society and American Association of Sleep Medicine Guidelines

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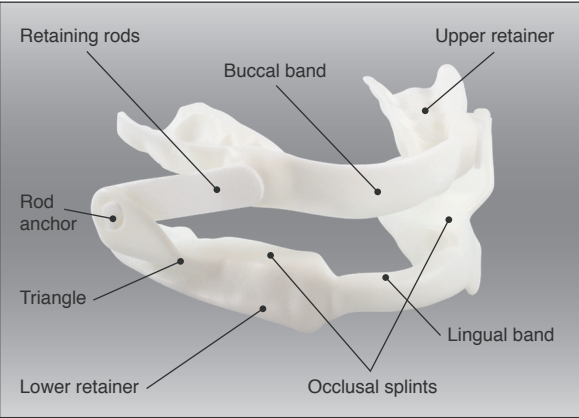
YOUR ANTI-SNORING DEVICE

The device maintains the lower jaw in a forward position during sleep. The advancement of the lower jaw clears the back of the tongue and facilitates opening of upper airways, which increases air flow.

The computer-aided design, the custom retainers, and the features of the device have a proven efficiency and offer increased comfort. The resistance of the material used enables a long term use with few, if any, side effects.

BEFORE SLEEPING ...

When you are sleeping, the throat muscles relax. The jaw and the tongue move backward, narrowing the upper airways.

The snorer must then make an effort to open his narrowed airways. The air flow is reduced and its forced passage creates vibrations in the soft structures (such as the uvula, the soft palate and the pharyngeal walls), which causes snoring.

60% OF MEN AND

40% OF WOMEN SNORE ¹

Loud snoring can wake up the bed partner and disturb his/her sleep. Generally, the spouse will adjust his/her bedtime schedule or sleep in a different room. Snoring alters mood, affects social relationships and diminishes the overall quality of life.

15% of men and 52% of women are affected by their spouse's snoring.²

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BONNE NUIT!

CONGRATULATIONS

We hope you will be satisfied with your new Panthera Anti-Snoring Device and the improvement of your overall quality of life.

Panthera Dental hopes you and your relatives will enjoy a restful sleep.

SLEEP WELL!

www.pantherasleep.com

L-001 rev07, Mars 2019

SUMMARY

THE PANTHERA ANTI-SNORING DEVICE

Information and Instruction Booklet