### Please read the instructions before using your Panthera D-SAD Classic or D-SAD X3.

#### Introduction

The D-SAD Classic and D-SAD X3 are devices for the treatment of snoring and mild to moderate obstructive sleep apnea (OSA) in adults. The device requires a prescription from a physician. The Panthera D-SAD devices are made to your precise measurements and are single patient use. How do these devices work? When worn during sleep, the devices are positioned on the teeth to move the lower jaw forward. By advancing the jaw, the tongue is also moved forward to help keep the airway open. Opening the airway eases the airflow, reducing snoring and OSA.

#### Indications for Use

The Panthera D-SAD devices are intended to reduce or alleviate snoring and mild to moderate obstructive sleep apnea (OSA) in adults.

#### Important to remember

Always wear both upper and lower splints together.

#### Before each use, check the device for:

- Possible damage: INSPECT your D-SAD for signs of damage, such as fractures and cracks. If the device is damaged or broken, the device must NOT be used any longer in order to avoid injuries, such as choking or laceration. Discontinue use and immediately contact your sleep dentist.
- Cleanliness: if the device is not clean, it should be cleaned as per the cleaning instructions.

#### Contraindications

The Panthera devices are contraindicated for patients who:

- Are under 18 years of age.
- Have loose teeth.
- Have advanced periodontal disease.
- Have severe respiratory disorders other than OSA.
- Have central sleep apnea.

#### Precautions

- The dental sleep specialist should consider the medical history of the patients, including history of asthma, breathing or respiratory disorders, other relevant health problems, and refer the patient to the appropriate healthcare provider before prescribing the device.
- The Panthera devices must be used during sleep at night or when taking a nap.
- Do not eat while wearing the device. It is possible to drink water.
- It is recommended to consult a dental professional for an annual checkup.

#### Possible Side Effects

The use of the Panthera devices may cause:

- Tooth movement or change in dental occlusion.
- Gingival or dental soreness.
- Pain or soreness of the temporomandibular joint.
- Obstruction of oral breathing.
- Excessive salivation or mouth dryness.
- Any abnormal incident related to wearing the device must be reported to Panthera Dental and for EU patients, must also be reported to the competent authority in your country of residence.
- Should you experience significant side effects, contact your dental practitioner.


#### Package Contents

- D-SAD Classic with rod kit OR D-SAD X3 with clip kit.
- Cleaning bath.
- Storage container.
- Instructions for use.
- Orthodontic elastic bands (optional for D-SAD Classic).
- Optional: Patient dental model.

#### D-SAD Classic

#### Parts

- Upper splint and lower splint with fixed pivot point (anchor).
- A rod kit for jaw advancement. The rods come in different lengths; the dentist will determine the best length for you.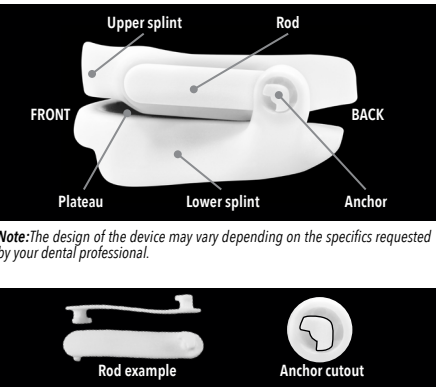


#### Rod Attachment


Attach the upper and lower splints (hinge) and allow to move the lower jaw forward.

- Attach one rod on each side of the lower splint into the anchor cutout, the fixed anchor point will match the cutout on the rod.
- Press firmly to secure. The rod number should be facing inward and the unattached end should be directed to the back of the splint.


Attach the upper and lower splints on one side first.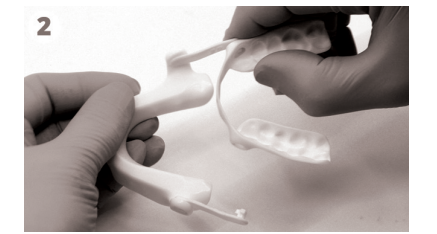

- To attach the upper splint, position it with the "teeth" side showing (facing upward), attach one side of the upper splint to the lower splint with the rod inserted into the cutout.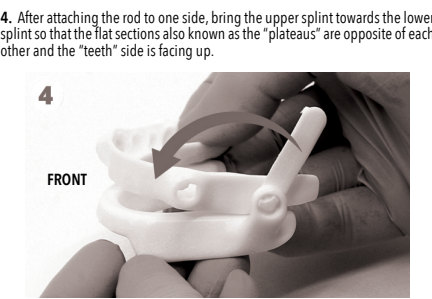

- To complete the second rod attachment on the upper splint, create a backward "L" with the device, and with a slight twist of the upper splint, insert the rod end into cutout on the anchor pivot point.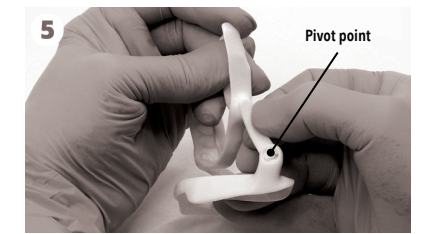


Once both rods are in place, the plateaus (flat sides) of the splints should be facing each other, and the D-SAD is ready to use.

#### Advancing the D-SAD Classic for Optimal Treatment

During the advancement phase, the rods may be changed by the dentist to obtain the optimum therapy position of the D-SAD Classic. The dental provider will inform you if this is needed. To increase protrusion, use shorter rods (ex: go from 26 to 25.5 mm). To decrease protrusion, use longer rods (ex: go from 25.5 to 26 mm). Once the final setting is achieved, the rods are fixed in place and do not need to be changed. The anchors and the rods become the permanent fixed mechanical hinge of the appliance. Always use the same length of rods on both sides, unless specified by your dental practitioner.

#### Removing the rods during the advancement phase

Recreate a backward "L" with the device as noted above (Figure 5). Once in this position, turn the rod to release from the upper splint anchor. Then remove the other side.

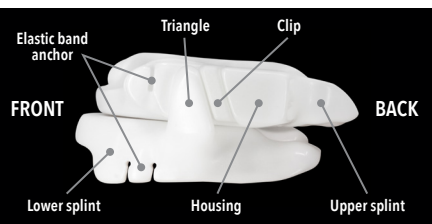
Note: The design of the device may vary depending on the specifics requested by your dental professional.

The D-SAD X3 is crafted with orthodontic elastic anchor slots on each side of the splint (upper and lower). To obtain the best treatment results, attaching the orthodontic elastic bands to connect the upper and lower splints is highly recommended.

- Attach one rod on each side of the lower splint into the anchor cutout, the fixed anchor point will match the cutout on the rod.
- Press firmly to secure. The rod number should be facing inward and the unattached end should be directed to the back of the splint.

Attach the upper and lower splints on one side first.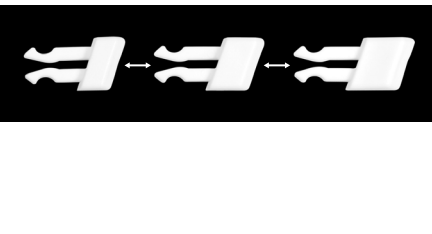

#### The next section applies to both devices

#### Getting ready for sleep

- Brush and floss your teeth and moisten your device with lukewarm water.
- As you place the device in your mouth, position the UPPER SPLINT FIRST on the upper teeth by pressing on both sides with your thumbs.
- Move your jaw forward, press the lower splint in place onto your lower teeth. Press on both sides with your fingers.
- Slowly close your mouth. In this configuration, the jaw is at a forward position.

#### Removing the D-SAD Classic or X3

- Remove the lower splint: place the thumbs on both sides of the lower splint, near the gum in front at the level of lower canines, and lift both sides of the device at the same time.
- Remove the upper splint: place your fingers on both sides of the upper splint, near the gum, and pull down on both sides of the splint at the same time.
- Remove the device from your mouth. Clean it according to the cleaning instructions.

#### Cleaning and storing instructions

- Every morning after each use, rinse your device in lukewarm water. Clean it with a soft bristle toothbrush or denture brush. Brush all surfaces and sides.
- Inspect your device for cleanliness and damage.
- Dry your device thoroughly with clean paper or cloth towel.
- Before storing your device, make sure the storage container is clean and dry.
- Clean your device at least twice a week with an antibacterial chlorine-free denture cleaner solution. Follow the manufacturer's instructions. Soak and brush the device with the solution. Rinse and dry well with a clean paper or cloth towel.
- Store in the storage container with the lid off, at room temperature away from direct heat, humidity and light. Never store your device in a closed container when it is humid.
- For extra cleaning, clean your device at least twice a week with an antibacterial chlorine-free denture cleaner solution. Follow the manufacturer's instructions. Soak and brush the device with the solution. Rinse and dry well with a clean paper or cloth towel.


- Do not put your device in boiling water (100°C or 212°F).
- Verify that you have no allergy to the ingredients present in the antibacterial chlorine-free denture cleaning solution before using it.
- Do not use mouthwash to clean your device, this could stain and induce odor or taste.
- Do not use chlorine-based products, bleach, acetic acid, household cleaning products or alcohol based solutions.
- Note: Discoloration of the device may be a normal occurrence; this is due to the acidity of saliva. Some medicine, amalgam fillings, coffee, tea, cola, wine may affect the color of the device. This will not affect the performance or the warranty of the device.

Wear your device as instructed by your dentist. If you have any difficulty with your teeth in the morning:


- Your dentist may provide instructions for morning exercise to help re-align your bite after wearing the device, such as moving the chin backward, knead molars together and hold for 2-3 seconds (repeat 5-10 times).
- Eat breakfast or chew sugar-free gum; it will help your bite go back to normal.

- Drink only water while the device is in place.
- Air Travel: It is advised to pack the device in your carry on.
- Keep away from pets as they may think it is a "chew toy".

- Upper and lower splints.
- A snap-in clip kit for jaw advancement.
- Orthodontic elastic bands.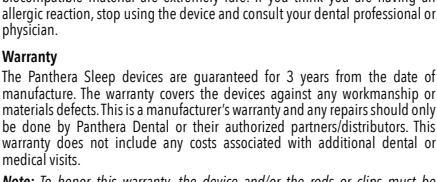


Note: The design of the device may vary depending on the specifics requested by your dental professional.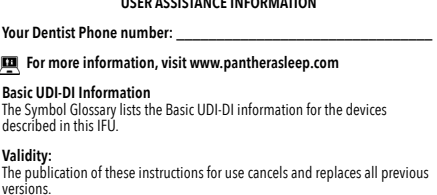

The D-SAD X3 is crafted with orthodontic elastic anchor slots on each side of the splint (upper and lower). To obtain the best treatment results, attaching the orthodontic elastic bands to connect the upper and lower splints is highly recommended.

- Attach one rod on each side of the lower splint into the anchor cutout, the fixed anchor point will match the cutout on the rod.
- Press firmly to secure. The rod number should be facing inward and the unattached end should be directed to the back of the splint.

Attach the upper and lower splints on one side first.

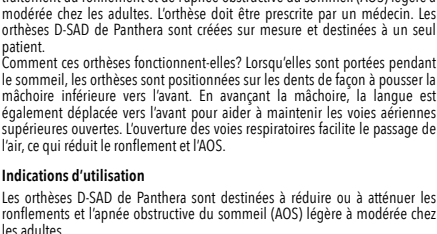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La parution de ce mode d'emploi annule et remplace toutes les versions antérieures. L'autorisation et la disponibilité des produits peuvent être limitées à certains pays / régions. © Panthera Dental, 2023. Tous droits réservés.