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Panthera Dental Milled Bar Instructions for use – Dental Practitioner Guide

This is not intended to replace or supersede sound medical judgement, the dental practitioner's experience or training. Panthera Dental does not provide medical advice.

Description of the device and expected performance: The Panthera Dental Milled Bar is intended for use as an accessory to an endosseous dental implant to support a prosthetic device in a partially or edentulous patient for purpose of restoring chewing function. It is intended for use to support multiple tooth prostheses in the mandible or maxilla. The prostheses can be screw retained. **Precautions:** Choosing the right bar according to the prosthetic project is the responsibility of the dental practitioner. **Safety, Responsibility and Warranty:** Bars provided by Panthera Dental are manufactured in accordance with European, American and Canadian standards for medical devices. No adverse effects are expected or reported. Physiological and anatomic conditions may negatively affect the performance of the device. **Storage, transportation and cleaning process:** Store in a dry and clean place at room temperature in the original box when possible. **Warning:** Do not damage the packaging while shipping. The product has no expiration date. The product is sold in a NON-STERILE packaging. It is the responsibility of the practitioner to clean/sterilize the device before use, according to the dental general practice: sterilization of the metal parts by following the standard medical procedures (i.e., steam autoclave sterilization). **Technical support:** Our technical staff is available to assist you with any questions regarding the use of the Panthera Dental bars. You can contact us by calling our call center. Additional information regarding the use of the bar is available in our main catalog and technical leaflets. **Maintenance and periodic care:** Dental practitioners have the responsibility to maintain the functionality and proper storage of the device, while ensuring the safety of the patient with consistent care. **Precaution:** For safe and effective use of device, it should only be installed by a trained professional. **Guideline for the patients:** Patients are recommended to follow the indications provided by the dental practitioner, go for periodic checks and maintain proper daily hygiene. **The kit includes:** Single bar with screws or attachments, if applicable.

TECHNICAL SPECIFICATIONS

Panthera Dental Milled Bar: Titanium bar to be screwed or fixed on endosseous implant.

Instructions for use:

Panthera Dental Milled Bar: Screw the bar to the implant with the proper screwdriver while making sure the insertion is correct. Screw tightly by using the proper dynamo-metrical drill extension tool.

In order to maintain the high quality standard offered by the Panthera Dental product line, an annual verification of the bar is recommended.

Any use of the milled bar which does not follow these instructions or other documentation provided by Panthera Dental is considered improper.

CAUTION: U.S. and Canadian Federal Law restrict this device to sale by or on the order of a licensed dentist or physician.