





Panthera Dental outdated 3Shape Library removal

To insure a proper working installation of our new library, please make sure to follow these instructions to remove all remaining trace of the old library installation. From the new version up, these steps will not be needed since the library ID will be the same.


1. Open the 3Shape **Dental System Control Panel**

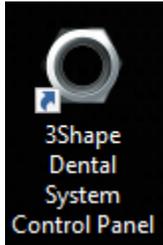


2. Open the **Implant Systems** of the **Abutment** section

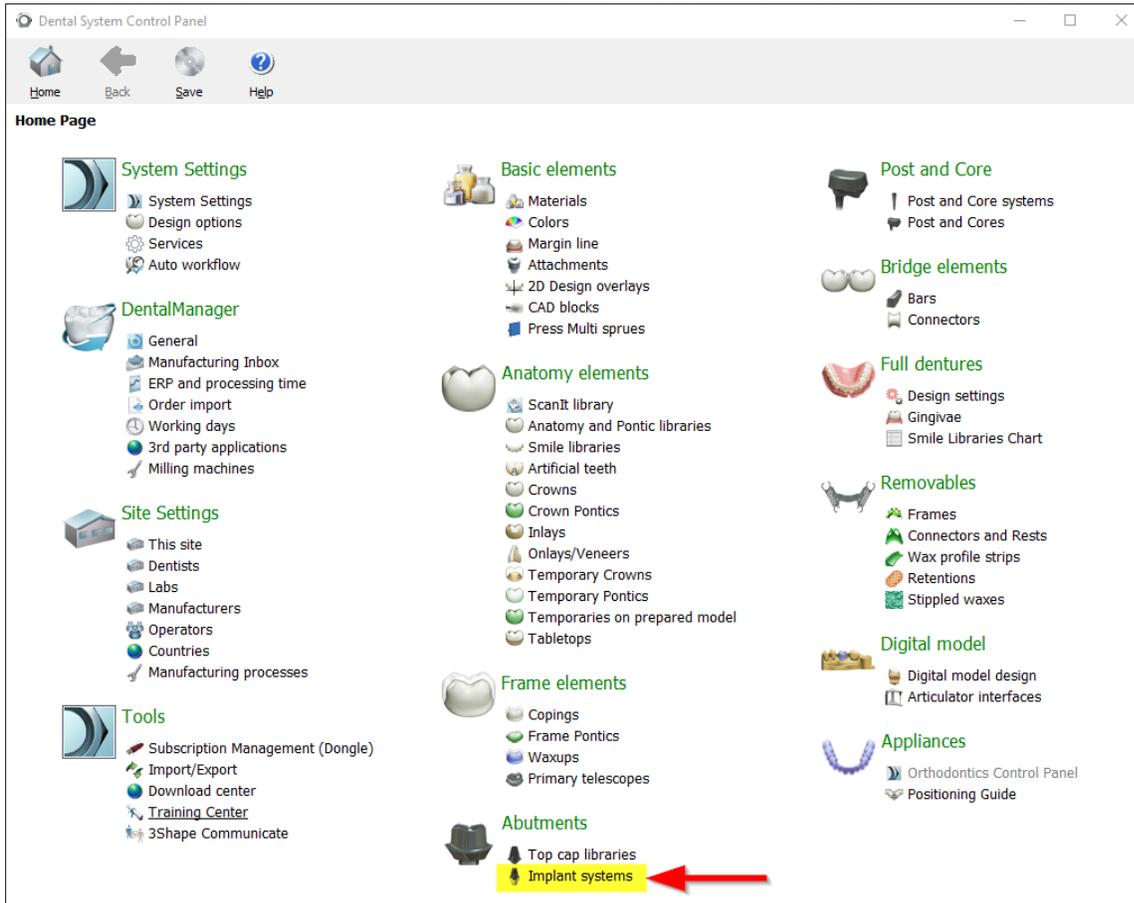


3. Delete all Implant system whose **Name** start with **“Panthera”** and who’s **ID** start with **“98004”** by selecting them and clicking on the **Delete** button. There should be about 26 of them.

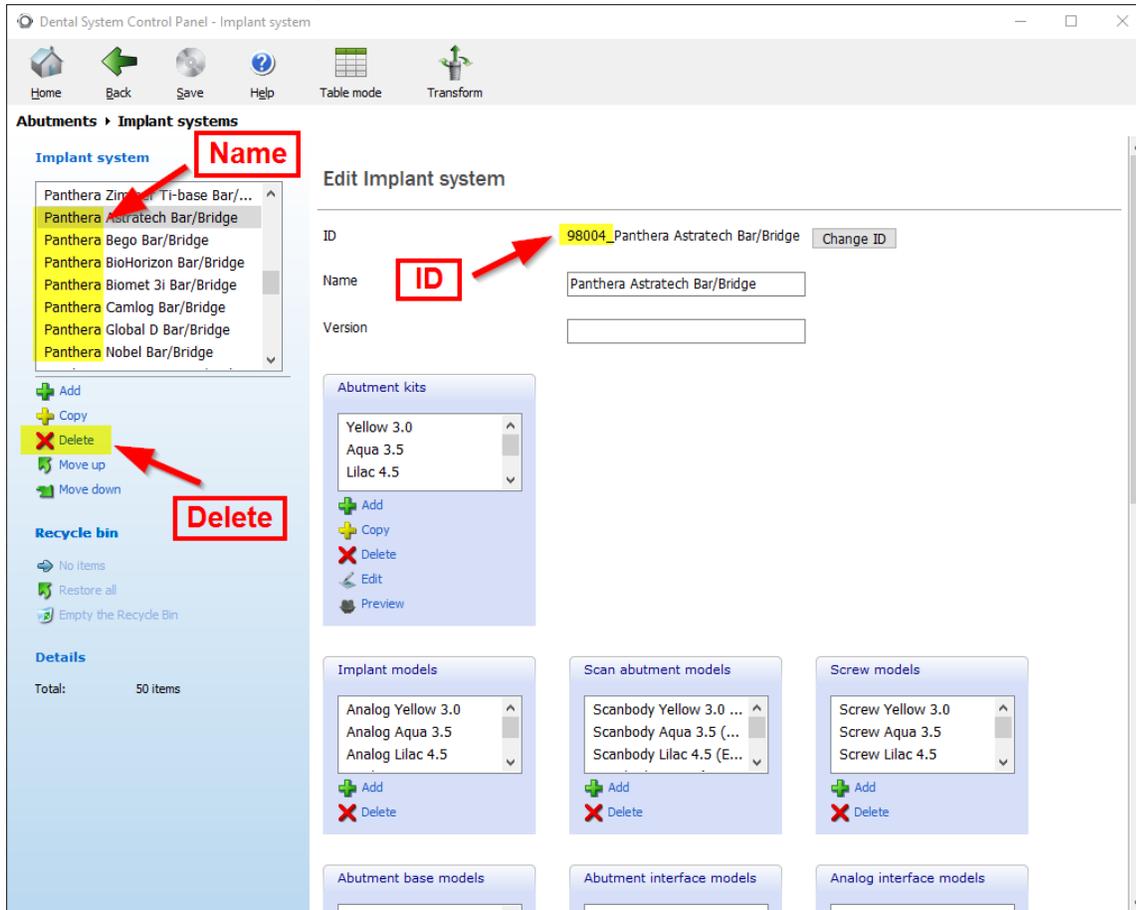


4. Back at home of the Dental System Control Panel by clicking on the **Back** or **Home** button on top left of the page.

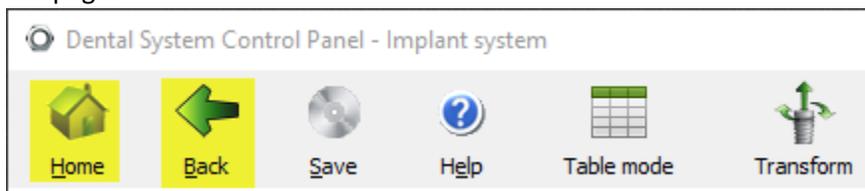


5. Open the **Materials** of the **Basic elements** section

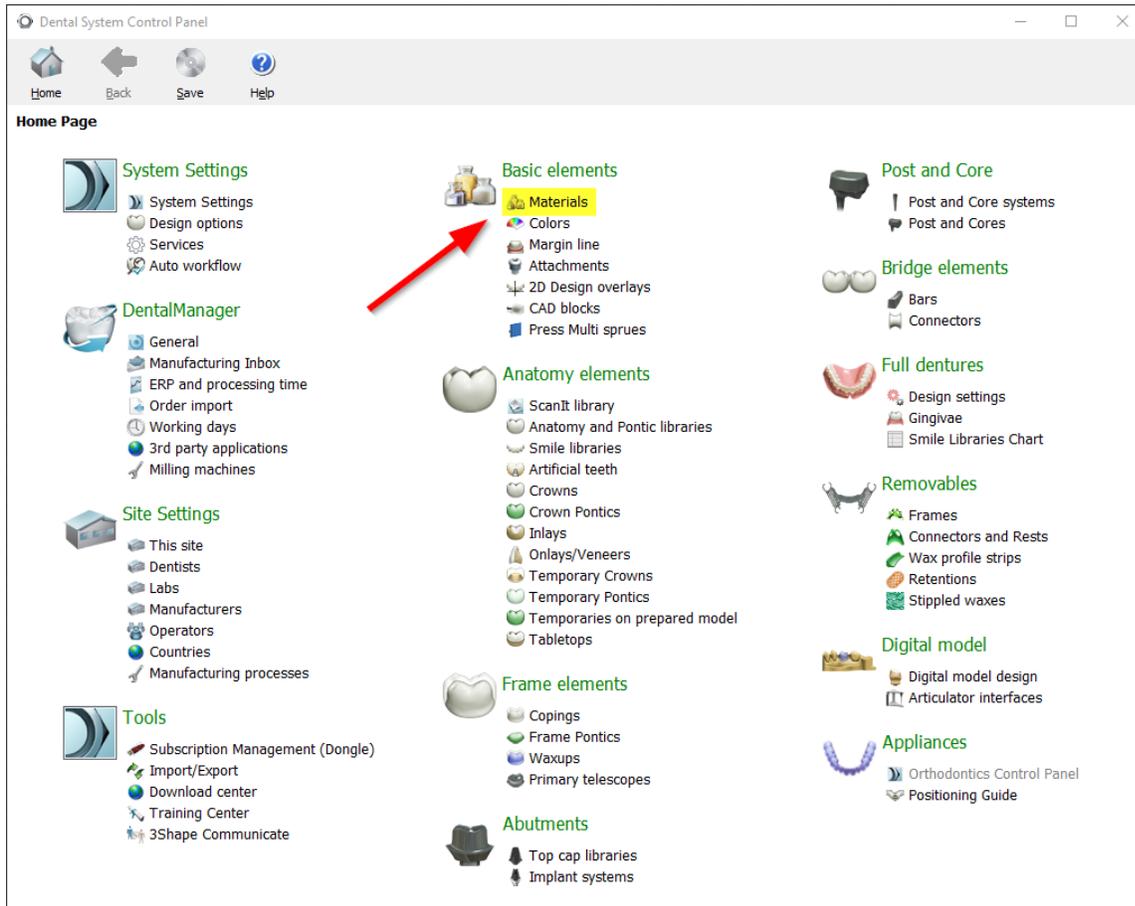


6. Delete all Materials whose **Name** start with **“Panthera”** and who’s **ID** start with **“98004”** by selecting them and clicking on the **Delete** button.

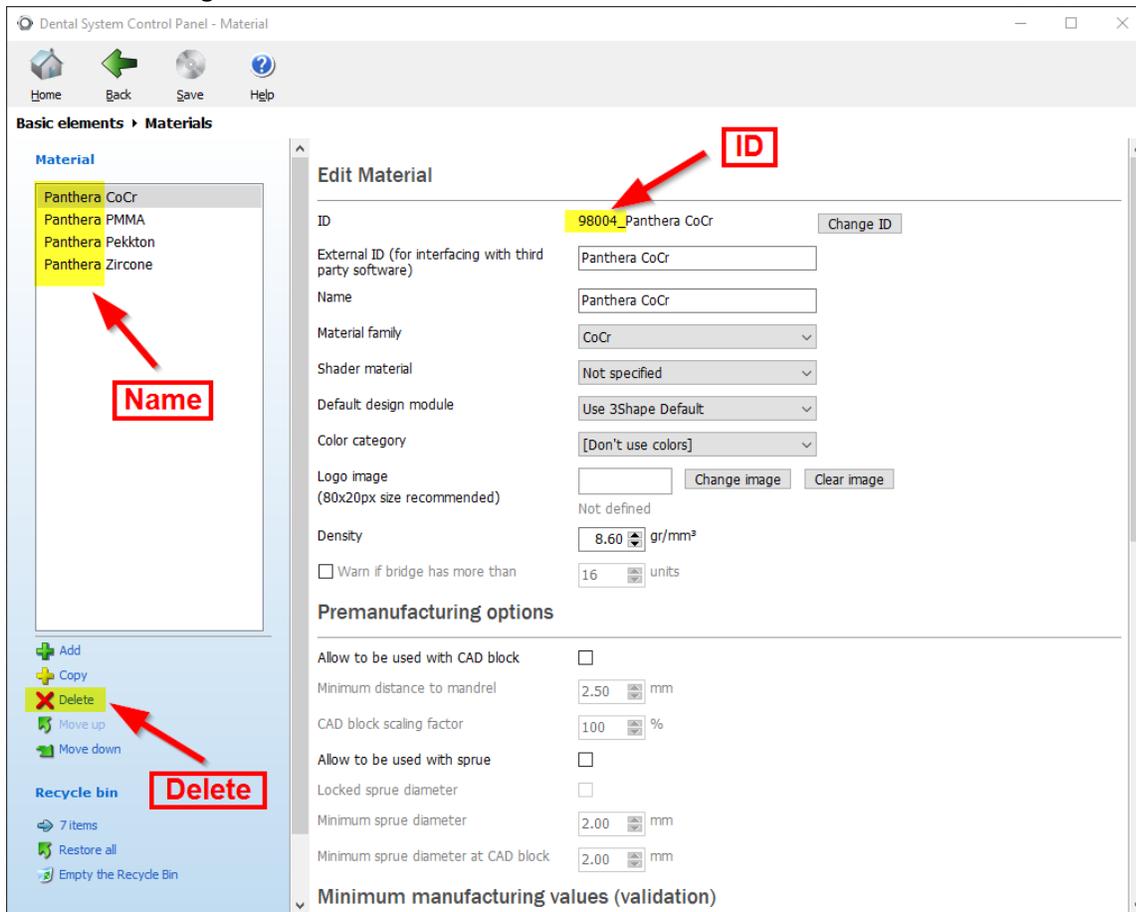


7. Back at home of the Dental System Control Panel by clicking on the **Back** or **Home** button on top left of the page.

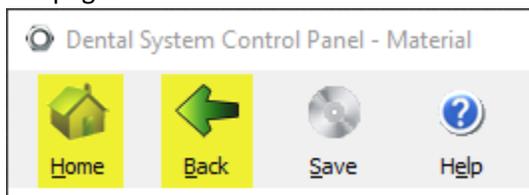


8. Open the **Manufacturing processes** of the **Site Settings** section

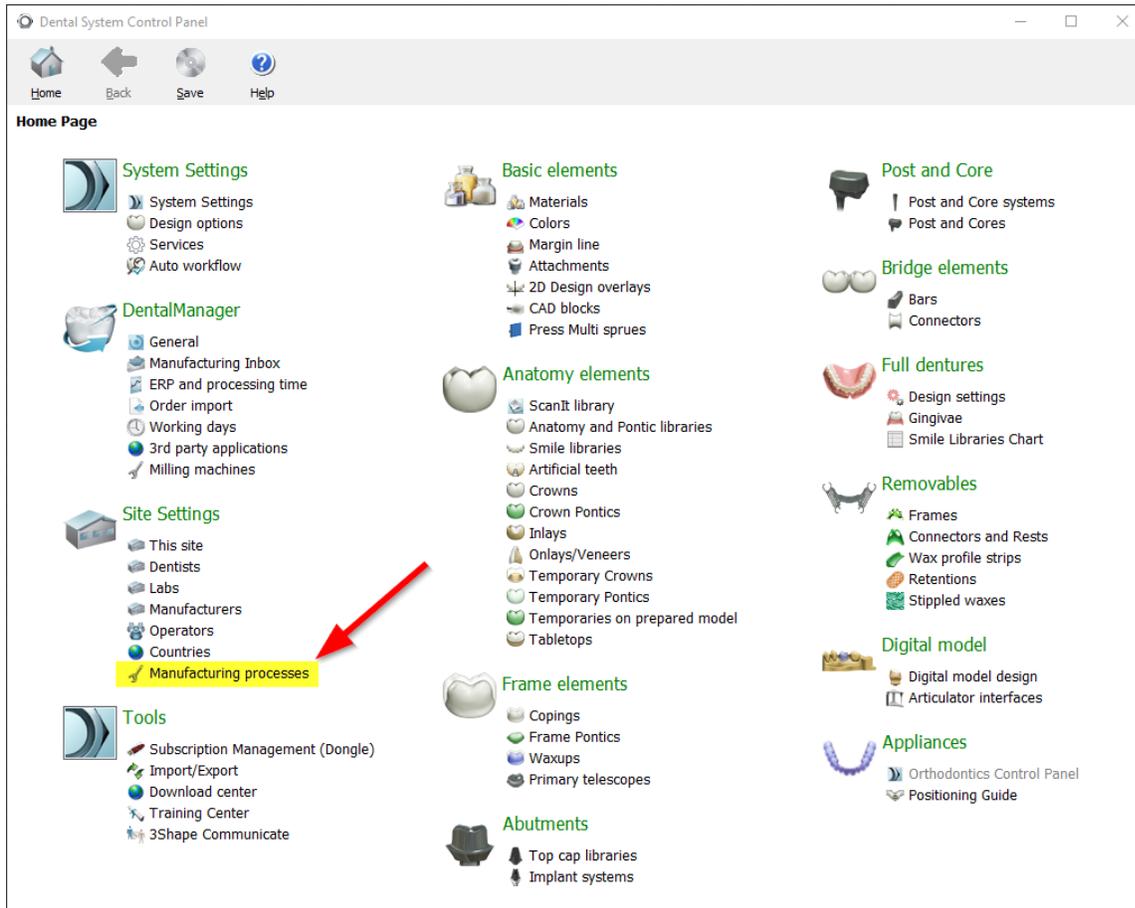

9. Delete all Manufacturing processes whose **Name** start with **“Panthera”** and who’s **ID** start with **“98004”** by selecting them and clicking on the **Delete** button.

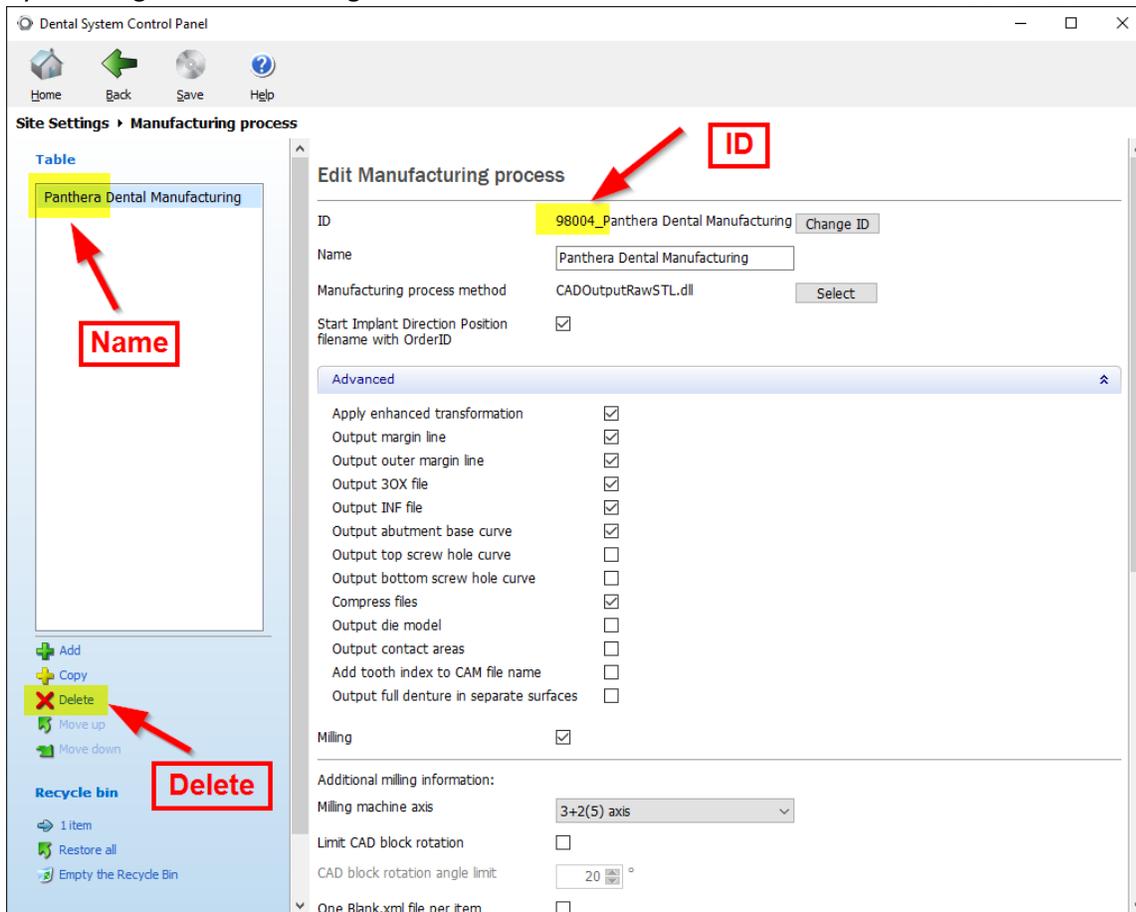

10. Back at home of the Dental System Control Panel by clicking on the **Back** or **Home** button on top left of the page.

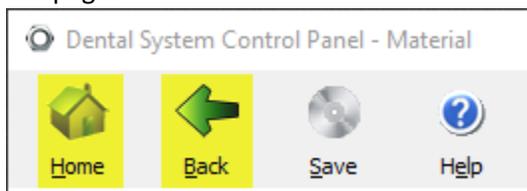

11. Save all changes by clicking on the **Save** button on top left of the page.

